



# ISOLUX

## **INSTRUCTION MANUAL**

**ASSEMBLY OF ADJUSTABLE HEIGHT MOBILE FLOOR STAND**



OM 3 RevB

# **ASSEMBLY INSTRUCTIONS:**

## **A. Illuminator with Adjustable Mobile Floor Stand**

The following instructions describe the assembly of the adjustable height mobile floorstand and swivel base. The mobile floorstand consists of three (3) parts:

- One (1) Five Legged Base with 2 locking and 3 rolling/non-locking Casters
- One (1) Adjustable Column Set
- One (1) 360° rotating, 5° tilted up Swivel with two (2) Headlight / Power Cord Hangers

Please follow the steps below in order to complete the assembly of the 30 inch floor stand to the 300 Watt Xenon Illuminator:



1. Place the Five Legged Base with Casters on the floor.
2. Adjust length of column to desired height at this time. Slide pin out and re-insert.
3. Slide the tapered end of the column into the center hole of the five (5) legged base.
4. Place the Illuminator upside down on a soft cloth or plastic and remove the four (4) knobs. The four (4) studs should face up.
5. Place the Swivel on the Illuminator above, so that the four (4) studs protrude through the slots of the swivel. Please note that the arrow on bottom of swivel indicates the front, to assure that the 5° tilt is positioned to the front panel of the Xenon Illuminator.
6. Tighten the four (4) knobs on the studs and make sure that the swivel and illuminator assembly are securely held together.
7. Turn the above assembly over and place the swivel column into the assembled base with casters.