



ISOLUX

There are two basic instructions for resetting the timers depending upon the serial # of your unit.

Model 1300XSB Premium Xenon Light Sources with serial #'s beginning with 3

- 1) After you have taken off the top cover and replaced the lamp, go to the timer connections on the inside of the unit
- 2) Find the wire connection to the junction on the timer labeled "I" (red one on the bottom) and move this connection to the junction terminal labeled "R".
- 3) Then turn the unit on for a few seconds, in which case this will reset the timer back to "0.0" hours.
- 4) Then turn the unit back off and replace the red wire back to it's original junction terminal labeled "I".
- 5) Your unit will then be reset and ready to be counting down the hours on your new lamp.

Model 1300XSB Xenon Light Sources with serial #'s beginning with 2

- 1) Remove the top cover from the light source
- 2) Notice on the top right side, where the power supply cover is, there is a small pin hole
- 3) Make sure that the unit is on for the timer to be reset
- 4) Using a q-tip or some type of swab stick that will fit into that hole, insert until you hit the switch that will reset the timer
- 5) You can also take off the power supply cover and push the switch manually if you cannot find anything to fit into the hole – switch is small and located next to the power switch on the interior of the unit.

1045 Collier Center Way
Suite 6-8, Naples FL 34110
Tel: +1 239 514 7475
Fax: +1 239 514 7476
Email: info@isoluxllc.com
Web: isoluxllc.com